VAXOL® EAR SPRAY

Vaxol® is a safe and effective method of softening and naturally removing ear wax. Thus reducing the need for syringing, or to facilitate syringing should it prove necessary.

Vaxol® is for the use of adults and children over I year old.

Vaxol® has a simple yet unique system that delivers a single, metered dose of the product into the outer ear. Easy and convenient to use. No over application significantly reducing spillage and mess.

Please read this leaflet before use.

BEFORE USING A NEW BOTTLE OF VAXOL® SPRAY

 Before using a new bottle of Vaxol®, warm the bottle in palm of hand, shake well, then press the actuator down 3 or 4 times until you get a spray (See Diagram b). When doing this point away from yourself. This procedure will prime the system and ensure the product is ready to use.

DIRECTIONS FOR USE

- Vaxol[®] is most effective when warmed in palm of hand and shaken well before use.
- Apply Vaxol® as directed by your Doctor or nurse
- Use once or two times a week to prevent ear wax building up and to help the self cleaning process
- If a medical procedure is required, apply 1 or 2 applications of Vaxol[®] twice daily for 4 or 5 days before visiting the doctor.
- Pull the upper ear or Pinna delicately upwards and backwards (See Diagram a)
- Holding the bottle in an upright position with the nozzle placed in the ear canal, depress the actuator fully 1 or 2 times (See Diagram b)
- Massage gently the Tragus (See Diagram a)
- · Repeat the procedure with the other ear if necessary
- · Do not rinse ear after use
- Vaxol® can be applied sitting or standing upright
- The actuator button can be removed and rinsed in hot water after use and then seated back on the stem.

ADDITIONAL INFORMATION

- Patients who have a perforated ear drum or ear pain should only use the product under medical supervision.
- Vaxol® should only be used for children under one year of age following medical advice.
- Ölive Oil is a natural product and is widely used. As such, no side effects and no interactions with other medicines have been reported with this product. However this product should not be used if you have a sensitivity or allergy to Olive Oil.

- There are no known harmful side-effects when using this product during pregnancy and/or breast feeding, however you should always inform your Doctor before use when pregnant or breastfeeding
- There are no known side-effects from overdosing with this product.
- If you feel you have experienced an allergic reaction stop using this product and inform your Doctor immediately.

WARNINGS

- Vaxol[®] is for single patient use only.
- Always store Vaxol[®] in an upright position at normal room temperature.
- The product should be discarded six months after first application.
- Dispose of the product as per Local Authority advice.
- · Do not spray near naked flame.
- · Keep out of the reach of children.
 - Vaxol[®] is only for use in the ear canal. Do not spray near eyes, nose or mouth.

Manufactured by: HL Healthcare Ltd

HL Healthcare Ltd 112-114 Witton Street Northwich Cheshire CW9 5NW United Kingdom

Distributed by: FARMASYN S.A.

6 Monis Damastas Street, 121 33 Peristeri, Athens Tel. 210 5777140 Fax 210 5788791

Vaxol[®] is a registered trademark of FW Medical Ltd. Leaflet last amended: June 2013.

Vaxol® is a medical device class I conforming to European Directive 93/42/CEE







VI24-I LAB0009I2